



HEARING LOOP INSTRUCTIONS FOR USE

- On the left-hand wall, below the shelf, is a row of 13 amp sockets and plugs.
- The plug at the back is marked 'Hearing Loop'—switch it on.
- The hearing loop amplifier ① on the top shelf will light up—allow approximately 15 to 20 seconds for it to go through its set up sequence.
- The front of the amplifiers digital display, will now show an oblong box, with the word 'LOOP' by the side of it. The amplifier is ready!
- On the right of the cupboard is a cardboard box marked 'hearing loop'. It's at about head height.
- Remove the assembly, microphone and power supply.
- On the top of the power supply is the microphone plug-in connection and a slide on/off switch. ②
- Slide the switch to the on position.
- Allow 5 to 10 seconds for the microphone to connect to the amplifier.
- The oblong on the amplifier will now show a series of black blocks moving backwards and forwards, dependent upon the noise being picked up on the microphone.

THE MICROPHONE IS READY TO USE EITHER AS A LAPEL ATTACHMENT OR PLACED ON A TABLE IN FRONT OF THE SPEAKER.

This is a hearing aid loop amplifier for digital hearing aids that automatically configure, or aids that can be switched to a 'T' setting. It is not an audience microphone for general use—that needs an independent system.

Please remember to turn the amplifier and microphone 'OFF' at the end of the session and return the equipment to the marked box.

